Ortec^{by EMO}

en

Dear customer, we appreciate the confidence placed in a EMO product. Choosing a brace of Ortec line you have purchased a quality product and high level medical doctor. Read these instructions carefully. If in doubt contact your doctor or retailer.

Regulations
The medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

Description and characteristics
Ortec RD600 knee immobilizer polyethylene: made of orthoses molded polyethylene bivalve adjustment for easy adjustment and knee immobilization.

Position the knee joint in slight flexion of 10°. Ortec RD400 controlled immobilizing knee brace is a velor tailored for its outer face and cotton toweling, on its inner side. It has three whale shaped duralumin that allow stabilizing the knee joint, inhibiting pain during scheduled rest periods. The adjustment of the orthosis is performed by velcro. The orthosis releases the ball. The material that is made orthotics allows easy adaptation.

Precautions
Follow the manufacturer's directions for proper orthosis placement. Follow the instructions or recommendations of the prescriber or healthcare personnel. For a personalized adaptation, consult a specialist. If there is ongoing pain, it is advisable to contact a healthcare professional. If an allergic reaction should occur remove the product and consult with the prescribing physician. Do not expose the garment to a flame or source of heat at very high temperatures. Store at room temperature, preferably in the original container. Warranty within the framework of laws.

Cleaning and maintenance
Hand wash with warm water

Directions
Injuries to knee -joint, ligament sprains, fractures, pre-operative and post-operative.

Instructions for use and installation
The product should be adapted by Orthopedic Associates or by the physician individually for each patient. The right choice of size is essential to achieve maximum effectiveness of treatment and prolong the life of the garment. The knee brace, by design, are easy to install. The patient can be removed and put the brace hardly mobilize the limb, facilitating rehabilitation exercises and personal hygiene. Adapt aluminum bars to the morphology of the patient. Place the brace on the area to the right height for use. Help of Velcro strips to close and adjust the garment. For the RD400 should first remove the side panels. After placing the patient's leg on the central panel at the correct height (align the patella to the level of the patellar hole of the garment. Subsequently secure the side panels according to the perimeter measure of the patient. Finally closing and adjusting straps through velcro.

(30 °) and mild soap. During washing, if the garment has Velcro, they must stick together or keep them closed. Dry at room temperature. No tent or iron the garment. Contact with ointments, creams, alcohols and solvents may damage the garment.

Quality Certificates
All materials used have been tested, comply with applicable European standards, are high quality and offer maximum safety and comfort during use. Oeko-Tex Standard 100: textile eco-label awarded after the completion of a catalog of criteria analysis that guarantees the absence of substances harmful to health. Certifies the absence of harmful substances in all materials with which a garment is made.

Notice
To dispose of the product, use the solid waste container. For the disposal of the container, depending on its material, use the paper / cardboard container or the plastic container. In any case, you must strictly comply with the regulations of the country. Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and / or the patient are established.

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