## HV200 Valguscontrol VAL100

en

Dear Customer, thank you for the trust placed in EMO product. Choosing this brace you have purchased a quality product, high level medical doctor. Read these instructions carefully. If in doubt contact your doctor or specialist dealer.

### REGULATIONS

The medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

### DESCRIPTION AND CHARACTERISTICS

VAL100 night orthosis for hallux valgus. Night corrective brace for valgus alignment of the big toe. Anatomical conformation that wraps around the foot. Aligner incorporates splint. Padded area for 1 finger. Made of soft velor. Closing microvelcro. HV200 night valgus correction of hallux valgus Control: Corrective night splint to align valgus the big toe. Anatomical conformation. It incorporates cushioning pad for 1 finger. Metatarsal on the 5th metatarsal regulated control pressure corrective strap. 1st metatarsal cinch on anatomical and retrocapital metatarsal. It incorporates comfort pad on the 5th metatarsal. Made of thermoplastic and soft velor. Closing microvelcro.

### EFFECTS

Bracing VAL100 night for hallux valgus: promote gradual alignment of the deviation of the finger. Getting a correction or slowing the evolution of the deviation of the first toe valgus and varus of the first metatarsal. HV200 night valgus correction of hallux valgus Control: The application of low intensity forces exerted by the orthosis by three support points placed the big toe as its axis correcting the valgus (bunion) and getting the metatarsophalangeal alignment. HV200 night valgus correction elongator Control generates a significant effect on the flexor and extensor muscles of the great toe. This effect, based on a constant low pressure and controlled intensity favors the gradual alignment of the deviation of the finger, reducing the bunion and achieving maximum standardization of tissue and joint.

### INDICATIONS

VAL100 night orthosis for hallux valgus: conservative treatment of

hallux valgus. Bunion night aligner HV200 night valgus correction of hallux valgus Control: conservative treatment of hallux valgus. Alignment Bunion night. Post-operative hallux valgus

### INSTRUCTIONS FOR USE AND INSTALLATION

VAL100 night orthosis for hallux valgus: 1 Release and fully open the Velcro brace. 2 Match 3 bunion splint Embracing the entire forefoot and close the Velcro on the dorsal area.

3 Embracing the big toe to the distal extension of the brace. 4 Readjust the central Velcro to the optimum pressure. HV200 night valgus correction of hallux valgus Control: 1 Release and fully open the Velcro brace. 2 Enter your thumb in the padded area of the splint. 3 gently place the metatarsal of the big toe strap behind the head of the first goal (bunion) and adjust. 4 Place the metatarsal strap on the 5th metatarsal (which incorporates adjustable meter) and insert it through the slot of the splint. After pulling until optimal alignment correction and tolerated by the patient. Incorporate comfort pad pressure on the area. The meter of this webbing facilitates the monitoring and progress of treatment based on a constant low pressure, tolerable and controlled intensity.

### PRECAUTIONS

Follow the manufacturer's directions for proper orthosis placement. Follow the instructions or recommendations of the prescriber or healthcare personnel. For a personalized adaptation, consult a specialist. If there is ongoing pain, it is advisable to contact a healthcare professional. Excessive compression can cause intolerance and cessation of treatment we recommend to regulate its optimum compression. The correct choice of the appropriate size for each patient is essential to obtain the highest degree of therapeutic efficacy and prolong the life of the brace. The adjustment of the brace must be individual. The use of this brace is conditional on the prescribing instructions for what should be used for the purposes indicated by him. The material used is hypoallergenic but if an allergic reaction or irritation occur withdraw the product and consult with the prescribing physician. Save the orthosis at room

temperature, preferably in the original packaging.

### WASHING AND MAINTENANCE

Hand wash in warm water (maximum 30 °) and neutral soap. During washing, if the garment has Velcro, they must be glued together or keep them closed. Dry at room temperature. Do not store or iron the garment. Contact with ointments, creams, alcohols and solvents may damage the garment. Do not use bleach or dry clean. To lay the garment dry and not exposed directly to a source of direct heat.

### QUALITY CERTIFICATES

All materials used have been tested, comply with applicable European standards are of high quality and offer maximum safety and comfort during use. Öko-Text Standard 100 textiles eco-label awarded after the completion of a catalog of criteria for analysis that guarantees the absence of substances harmful to health. Certifies the absence of harmful substances in all materials that a garment is made.

### WARRANTY

Orthopedic Medical Specialties si guarantees all its products as long as they have not been tampered with or altered and always within the framework and terms of the laws. The products whose characteristics have been altered due to misuse are not included within this warranty.

### NOTICE

To dispose of the product, use the solid waste container. For the disposal of the container, depending on its material, use the paper / cardboard container or the plastic container. In any case, you must strictly comply with the regulations of your country. Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and / or the patient are established.

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