Lumback flex

Dear customer, we thank you for your trust in the products of EMO. By choosing an orthotic of the Lumback Flex line you have acquired a qualitative medical product. Read these instructions carefully. If you have any questions, please contact your doctor or your specialist.

NORMATIVE

The medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

DESCRIPTION AND CHARACTERISTICS

FJ100 Lumbosacral orthotic: sacrolumbar orthotic is made with elastic fabric that is highly breathable. It is an elastic sacral ligament that supports the body. It is incorporated with four flexible bands. Due to the manufacturing without sewing, overpressure is prevented and it makes the products as light as possible. Two large loops for easy placement and it could be closed with micro-Velcro. Latex free: hypoallergenic. FJ200 Lumbosacral orthotic: sacrolumbar orthotic made with elastic fabric that is highly breathable. An inelastic sacrolumbar orthotic that supports the applicable area. It is incorporated with four flexible bands. Due to the manufacturing without sewing, overpressure is prevented and it makes the products as light as possible. Two large loops for easy placement and it could be closed with micro-Velcro. Latex free: hypoallergenic. FJ151 Lumbosacral Orthosis Adjustable Work: Extremely breathable: comfortable to use for long periods. Elastic bands for greater sacrolumbar relief. It is incorporated with four flexible bands on the back and the Velcro makes it easy to close and adapt the orthosis. It is incorporated with straps that help to hold the girdle over the shoulders, when the patient does not need to wear it tight.

EFFECTS

FJ100: fastening, lumbar discharge, mobility control and

proprioception. FJ200: fastening, lumbar discharge, lumbar support, mobility control and proprioception. FJ151: fastening, labour protection and Lumbar support.

INDICATIONS

FJ100 and FJ200: Acute and exceptional low back pain, Facet syndrome, disc protrusion. FJ151: Low back pain, mild trauma, muscle hypotonia. It protects patients during activities that force a continuous overload of the lumbosacral zone and for possible injuries.

INSTRUCTIONS

FJ100 and FJ200: place the orthotic on the sacrolumbar zone at the correct height. Use your fingers to tighten the orthotic over the abdomen. Close the orthotic with the Velcro once obtained the desired compression. FJ151: Fasten the short ends of the straps to the back of the strap (in case it is necessary to use them). Place the orthotic on the sacrolumbar area at the correct height. With the loose sidebands the orthotic can be tightened over the abdomen. Close the orthotic with the Velcro once obtained the desired compression. Regulate the pressure with the help of the lateral bands and fix them on the abdominal area. By using the tie rods, fix the adjustable ends in the front and regulate the pressure by using the buckle.

PRECAUTIONS

Follow the manufacturer's directions for proper orthosis placement. Follow the instructions or recommendations of the prescriber or healthcare personnel. For a personalized adaptation, consult a specialist. If there is ongoing pain, it is advisable to contact a healthcare professional. The used material is hypoallergenic but if there is any allergic reaction or irritation remove the product and consult with the prescribing doctor. In case of discomfort caused by excessive sweating we recommend to wear a cotton t-shirt. The user should position

itself in such a way that the compression is not excessive. In case of doubt about the size, choose the bigger size. Only use during active periods. Reduce abdominal pressure during daytime breaks or after meals. Remove during the night. The correct choice of the appropriate size for each patient is essential to obtain the highest degree of therapeutic efficacy. The adjustments of the orthotic should be made individually. The orthotic should be used for the purpose it is prescribed for. Do not expose the orthotic to a flame, heat source or high temperatures.

WASHING AND MAINTENANCE

Hand wash with warm water (maximum 30 degrees) and neutral soap. The Velcro on the products must be kept closed during the washing. Dry at room temperature. Do not iron the garment. Contact with ointments, creams, alcohols and solvents may deteriorate the garment. Do not use bleach or dry clean. Do not expose the orthotic to a direct heat source by drying. Do not expose to a direct heat source. Dry and store the product at room temperature, preferably in the original package.

CERTIFICATES

All the used materials have been tested, comply with the applicable European regulations, are of high quality and offer maximum safety and comfort during use. Öko-Text Standard 100 Certified: awarded with the Textile eco-label by completing a catalogue of criteria that ensure the absence of harmful substances. Certifies the absence of harmful substances in all materials with which the orthotic is made.

WARRANTY

Especialidades Médico Ortopédicas S.L., guarantees that all its products, as long as they have not been manipulated or altered, always are within the framework of the legal terms.

Products whose characteristics have been altered due to misuse are not included in this warranty.

NOTICE

To dispose of the product, use the solid waste container. For the disposal of the container, depending on its material, use the paper / cardboard container or the plastic container. In any case, you must strictly comply with the regulations of the country. Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and / or the patient are established.

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