EMO
especialistas
en ortopedia

RD 2500 / RD 3000

Dear Customer, thank you for the trust placed in EMO product. Choosing this brace you have purchased a quality product, high level medical doctor. Read these instructions carefully. If in doubt contact your doctor or specialist dealer.

REGULATIONS
The medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

DESCRIPTION AND CHARACTERISTICS

RD2500 stabilizing knee brace with monocentric hinge: adjustable and extensible monocentric joint: flexion and extension of the knee controlled at different angles. Rapid locking system of 0, 15 and 30. Flex Control 0 ° to 120 ° in 10 ° intervals. Controlling the extent of 0 to 90 at 10 ° intervals. Telescopic extension system of the joint: minimum height of 46 cm. Maximum height 69 cm. Middle anterotorsorial side plates. Fastening straps (4) with quick clamping system and independent adjustment. Breathable, non-slip pads. Full front opening.

Ortec RD3000 functional knee stabilizer: Open design, which adapts to the thigh and calf, allowing a simple, custom adjustment by straps seal with tip and removable to facilitate adaptation. Set of four points of support. Aluminum frame gives stronger alloy and lightweight. Anatomically preformed tapes and suitable fastener. Lightweight design. Adaptable to patient alloy frame. Bagas adjustable.

cm), L (53-60 cm), XL (60-67 cm), XXL (> 67).

EFFECTS

RD2500 stabilizing orthosis knee joint monocentric: It stabilizes the knee joint. Medio-lateral control. Anteroposterior control. Control the degree of flexion and extension of the joint: immobilization of the joint or range of motion control. Limited potential pathological movements.

INDICATIONS

RD2500 stabilizing knee brace with monocentric joint: knee injury. Knee instability. Knee ligament injury. Post-traumatic rehabilitation. Post-operative rehabilitation.

Ortec RD3000 Stabilizing functional knee brace is indicated for injuries trataminato ACL, PCL, MCL and LCL, post-traumatic and post-operative rehabilitation. Ideal for prophylactic use.

Recommended for sports.

INSTRUCTIONS FOR USE AND INSTALLATION

RD2500 stabilizing orthosis knee joint monocentric:
1 Release and fully open the Velcro brace. 2 fully extend the brace. The joints are marked with "TOP" at the end to the thigh area. Place the patient supine with the leg between the two joints. 3 Adjust the length of the two joints (properly aligned relative to the axis of the lower limb and with respect to the physiological joint). The length of the two joints must be the same. 4 loosen the extero area of the straps and fasten the buckle. Adjust the length of the straps to the perimeter of the patient's leg. 5 If the length of the straps is excessive remove terminals velcro type "Y", trim excess and reseal the terminal. 6 Adjust the straps and fasten with Velcro. Ortec RD3000 placement of functional knee Stabilizer Attach bottom (calf) and upper belts (thigh). Keep in back and joint injuries position.

ACL: close the rear-lower thigh strap, extend joints, tibial strap closure, close the rear calf strap PCL: close the rear calf strap, extend joints.

Cf: close the front thigh strap. Cf: close the rear-lower thigh strap, close the upper calf strap, extend joints, tibial strap closure,

close the lower thigh strap Close the front thigh strap Ask the patient to flex the knee several times, even a few steps to make sure the knee does not slip and that the joint is in place. Adjust the straps if necessary.

Adjusting joint: 1) Lift the cover of the joint by removing the screws. 2) Place the piece of limitation of extension (default 10 °) as prescribed. 3) Place the piece of limitation of flexion as prescribed. 4) Place the lid making sure the screws holding the parts of the garment is made.

WARRANTY

Orthopedic Medical Specialties sl guarantees all its products as long as they have not been tampered with or altered and always within the framework and terms of the law.

The products whose characteristics have been altered due to misuse are not included within this warranty.

NOTICE

To dispose of the product, use the solid waste container. For the disposal of the container, depending on its material, use the paper / cardboard container or the plastic container. In any case, you must strictly comply with the regulations of the country. Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and / or the patient are established.

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en

REGULATIONS

The medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

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WASHING AND MAINTENANCE

Hand wash in warm water (maximum 30 °) and neutral soap. During washing, if the garment has Velcro, they must be glued together or keep them closed. Dry at room temperature. Do not store or iron the garment. Contact with ointments, creams, alcohols and solvents may damage the garment.

Do not use bleach or dry clean. To lay the garment dry and not exposed directly to a source of direct heat.

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