Dear customer, we appreciate the confidence placed in a product EMO. Choosing an Ortec line brace you purchased a quality product and high level medical doctor. Read these instructions carefully. If in doubt contact your doctor or your specialist dealer.

REGULATIONS

Product in accordance with the European Directive on Sanitary Products 93/42/CEE (Royal Decree 1591/2009). A risk analysis has been performed prEN 1441 (UNE EN ISO 14971) minimizing all possible risks.

DESCRIPTION AND CHARACTERISTICS

The modular system of hand is based on two pieces, one for the hand and one for the forearm, which combines posibilidades.Férulas multitude of mounting simple and easily adaptable, adjustable, conformable, adjustable, progressive and which can evolve as the patient improves.

BASE SMM201 FOREARM (SIZES: S, L, SIDES: RHS, LHS) SMM301 Palmer Station (SIZES: S, L, SIDES: RHS, LHS) BASE SMM101 FALANGE (SIZES: S, L)

For connection between palmar and forearm databases have several possibilities for the wrist joint.

SMM001 Elbow Joint (F-E) JOINT SMM002 DORSAL (CU-RA) WRIST LOCK SMM003 PROGRESSIVE SMM021 Elbow Joint (F-E) PROGRESSIVE RIDGE JOINT SMM022 (CU-RA)

For action on the fingers in flexion or extension, and even lateral deviation, we can use several options. SMM004 THOMAS (3 pieces) THOMAS SMM005 ALTO (3 pieces) THOMAS BRACE SMM006 FINGER FLEX SMM007 PLASTIC FINGER PLATE BENDING SMM077 THUMB ABDUCTOR SMM008 THUMB STABILIZER SMM010 (SIZES: S, L, SIDES: RHS, LHS) SMM020 SPRING THOMAS (SIZES: S, L)

The hardware necessary for the unions. SMM309 SCREW M4 (0 LONG SHORT) (25 bags UD)

Accessories for the orientation of the fingers and the tension required SMM030 finger cot (BAG 10 UNITS) Fingerstalls SMM040 DRIVE (SIZES: S, M, L) ELASTIC RINGS SMM050 (BAG 25 UD) 06,08,10 or 12 ELASTIC SMM060 CMS (BAG 100 UD)

DIRECTIONS

The modular system is a set of hand pieces combined together allow the technician to perform active splints, dynamic postural treatment, mainly rehabilitation of traumatic or postsurgical hand, thus early treatment of immediate application, with multiple combinations easily adjustable and adaptable.

INSTRUCTIONS FOR USE AND PLACEMENT

All pieces are placed with the screws 309 and allow adjustment of the hardware through extendable slot. The length of the elastic force us to define the action and must be determined by the physician.

PRECAUTIONS

Ready-made orthoses should be used strictly following the instructions of the physician or technician. Do not exert excessive pressure on the adjustment of the tape to avoid choking the hand or forearm. Should there be an allergic reaction to remove the product and consult with the prescribing physician. Store at room temperature, preferably in the original packaging. Guarantee within the framework of the laws. The fit of the brace should be individual.

WASHING AND MAINTENANCE

To tape adjustment: Hand wash with warm water (maximum 30 °) and mild soap. During

washing the velcro should be glued together or keep them closed. Dry at room temperature. Do not store or press. Contact with ointments, creams, alcohol and solvents can damage the material. Do not bleach or dry clean. To lay the garment dry and not exposed directly to a heat source. For housing and metal parts of the brace, wash exterior and interior surfaces with a cloth moistened with alcohol.

QUALITY CERTIFICATES

All materials used have been tested, comply with applicable European standards, are high quality and offer maximum safety and comfort during use. Oko-Tex Standard 100: textile eco-label awarded after the completion of a list of criteria of analysis that guarantees the absence of substances harmful to health. Certifies the absence of harmful substances in all the materials that is made of a garment.

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