PS100 / PS200 / PS300 Fasciplant / PS600 / PS650 / PS900

en

Dear Customer, thank you for the trust placed in EMO product. Choosing this brace you have purchased a quality product, high level medical doctor. Read these instructions carefully. If in doubt contact your doctor or specialist dealer.

REGULATIONS
The medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

DESCRIPTION AND CHARACTERISTICS
PS100 Ortec antidecubital heel is a piece of soft and pleasant to touch material heel protection. One size. PS200 Ortec postural antidecubital, is a design that frees heel, malleoli and toes weight of the sheets. It provides comfort and helps to maintain posture, absorbing pressure surges that may occur on the heel. It maintains the neutral position of the ankle, and allows you to adjust up to 15° internal / external rotation of the foot. One size. PS300 Ortec fasciplant, night postural splint for the treatment of plantar fasciitis. Double elongator system acting on the plantar fascia and the extensor muscles of the foot. Light anatomical conformation and design. Fastening straps on tarsus, malleolus and calf. Sub-phalangeal for maximum elongation bilateral wedge. Slip soles. Made of thermoplastic. Soft lining and pads for maximum comfort.

Microvelcro closing. One size with adjustable from # 34 to # 45 shoe. PS600 Ortec multi-position design with multi-position capability ankle angulation (equino 5° / 11th / 18th / 25th - neutro 0° - talo 5th / 11th). Antidecubital for its cushioning, primarily in the area of his great relief heel and toes for its plant that prevents contact with the sheets height. Allows the regulation of internal / external rotation of the foot (+ 15° / 0° / - 15°). Splint for correct placement of the lower extremity in bedridden patients. Sizes (no shoes): S (<32); M (33-38); L (39-43); XL (> 43)

PS650 Ortec dynamic multi-position device is a kit of parts that allows us to convert a static splint with multiple positions in a dynamic splint. It comprises two extensible slotted three Velcro strips of adhesive and an elastic band. PS900 Ortec leg posture, structure stability for its wide base of support. Fully padded with foam formed pieces possible bedsores. The plant triangle position, the brace stabilizes and allows internal / external rotation of the assembly 15th or neutral. Sizes: normal (Knee) long (above knee).

EFFECTS
PS100 Ortec antidecubital heel dampens overpressure in bedridden patient. PS200 Ortec antidecubital

postural, prevents scar formation by releasing the pressures in patients bedridden for long periods of time.

PS300 Ortec fasciplant: elongation of the plantar fascia. Achilles tendon elongation. Elongation of the extensor muscles of the foot. Dorsiflexion.

PS600 Ortec multi-position, for correct placement of the lower extremity in bedridden patients with multiple angular positions of the ankle and internal / external rotation.

PS650 Ortec dynamic multi-position device, allowing movement of the ankle dynamically. PS900 Ortec postural leg, allows the rest of the lower extremity in the position of rotation indicated by the doctor. The stability of the ferule is given by the large base and the place of the triangle planting position.

INDICATIONS
PS100 Ortec antidecubital heel, prevention of decubitus ulcers in patients.

PS200 Ortec postural antidecubital, preventing bedsores in the superficial tissues of the ankle. Pressure relief in patients bedridden for long periods of time. PS300 Ortec fasciplant: plantar fasciitis. Achilles tendinitis. Calcaneal spur.

PS600 Ortec multi-position leg, after adjusting the position of the triangle as indicated by the optional lower extremity introduce checking that the distal part of the finger does not stick out of the plantar fasciitis. PS900 Ortec postural leg, postoperative hip, knee and foot where the positioning of the lower extremity, antidecubital to its cushioning, particularly in the area through the gap formed in the foam and fingers need to finish adjust. Loosely padded straps that are placed in the eyelets on the housing side.

PRECAUTIONS
Follow the manufacturer's directions for proper orthosis placement. Follow the instructions or recommendations of the prescriber or healthcare personnel. For a personalized adaptation, consult a specialist. If there is ongoing pain, it is advisable to contact a healthcare professional. The used material is hypoallergenic but if there is any allergic reaction or irritation remove the product and consult with the prescribing doctor. The user should position itself in such a way that the compression is not excessive. In case of doubt about the size, choose the bigger size. Only use during active periods. Remove during the night. The correct choice of the appropriate size for each patient is essential to obtain the highest degree of therapeutic efficacy. The adjustments of the orthotic should be made individually. The orthotic should be used for the purpose it is prescribed for. Do not expose the orthotic to a flame, heat source or high temperatures.

INSTRUCTIONS FOR USE AND INSTALLATION

PS100 Ortec antidecubital heel, place the garment so that the heel is centered, close the straps to prevent movement of the garment without exerting excessive pressure. PS200 Ortec antidecubital postural, place the splint so that the flat base of the triangle rests on the bed. Set the degree of rotation, according to the required position by tightening the screws on the guides. Place the foot of the patient (superior) position adapted to the shape of the splint. Adjust the two Velcro: the more proximal must maintain a relative immobilization of the ankle; and the other should be used with the back foot avoiding possible movements. Fasciplant PS300 1. With the patient supine set above ankle strap supporting posterior calcaneus 2. Place before the velcro wedge, causing the extension of the anterior tibial 3. Set MTF 4. Adjust forefoot strap webbing. 5. With the lateral adjustment straps with bilateral ankle joint position 90 or cause talo elongando twins and Achilles. Fasciplant can be adjusted directly on the skin of

the patient by padding, or the patient may be a protective or sweat absorbent sock. Fasciplant is a brace for night use and short trips can support it by their slip soles.

PS600 Ortec multi-position, for placement of the brace, adjust the angle of the ankle strap on the position indicated by the specialist. Place the triangle at the base position, entering from inside the two screws that come with the triangle, closing with nuts from the outside. After adjusting the position of the ankle and the triangle position introduce lower extremity checking that the distal part of the finger does not stick out of the plantar fasciitis. PS650 Ortec dynamic multi-position device, replace the extendable grooved perforated originally carrying the splint. The three Velcro strips of adhesive are placed one on the outside of the plant and the other two outside the sides of the shank behind the top closure. These elastic velcro tape tension can be adjusted as necessary to notice. The device can be withdrawn at any time and can be exchanged for slotted extensible perforated and vice versa.

PS900 Ortec postural leg, after adjusting the position of the triangle as indicated by the optional lower extremity introduce checking that the distal part of the finger does not stick out of the plantar fasciitis. PS900 Ortec postural leg, postoperative hip, knee and foot where the positioning of the lower extremity, antidecubital to its cushioning, particularly in the area through the gap formed in the foam and fingers need to finish adjust. Loosely padded straps that are placed in the eyelets on the housing side.

INSTRUCTIONS FOR USE AND INSTALLATION

PS100 Ortec antidecubital heel, place the garment so that the heel is centered, close the straps to prevent movement of the garment without exerting excessive pressure. PS200 Ortec antidecubital postural, place the splint so that the flat base of the triangle rests on the bed. Set the degree of rotation, according to the required position by tightening the screws on the guides. Place the foot of the patient (superior) position adapted to the shape of the splint. Adjust the two Velcro: the more proximal must maintain a relative immobilization of the ankle; and the other should be used with the back foot avoiding possible movements. Fasciplant PS300 1. With the patient supine set above ankle strap supporting posterior calcaneus 2. Place before the velcro wedge, causing the extension of the anterior tibial 3. Set MTF 4. Adjust forefoot strap webbing. 5. With the lateral adjustment straps with bilateral ankle joint position 90 or cause talo elongando twins and Achilles. Fasciplant can be adjusted directly on the skin of

or iron the garment. Contact with ointments, creams, alcohols and solvents may damage the garment. Do not use bleach and short trips clean. To lay the garment dry and not exposed directly to a source of direct heat.

WASHING AND MAINTENANCE
Hand wash in warm water (maximum 30 °) and neutral soap. During washing, if the garment has Velcro, they must be glued together or keep them closed. Dry at room temperature. Do not store

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QUALITY CERTIFICATES
All materials used have been tested, comply with applicable European standards are of high quality and offer maximum safety and comfort during use. Öko-Tex Standard 100 textiles eco-label awarded after the completion of a catalog of criteria for analysis that guarantees the absence of substances harmful to health. Certifies the absence of harmful substances in all materials that a garment is made.

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