Lumbosacral brace pregnancy

Faja lumbosacra embarazo

FJ280

Belt brace pregnancy

Faja cinturón embarazo

FJ270

ES FJ270 / FJ280

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INSTRUCTIONS FOR USE AND PLACEMENT:

Place the brace over the pelvic area. Surround the abdomen with the belt by fastening the velcro straps until the desired degree of support is achieved. The front area must be positioned below the abdomen so that it fulfils its function of support. Subsequently, adjust the degree of compression and support by pulling the side tensioners forward at the same time.

EN FJ270 / FJ280

Dear customer, thank you for your trust in an EMO product. By choosing our line of orthoses, you have purchased a range of quality products of the highest sanitary standard. Please read these instructions carefully. If you have any questions, please contact your doctor or specialist retailer.

REGULATIONS (MD) This medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. A Risk Analysis (UNE EN ISO14971) has been carried out to minimise all existing risks. Class I medical device, non-sterile, without measuring function.

IMPORTANT INFORMATION Read all warnings and instructions before using this product. Proper application is essential for proper function and to reduce the risk of injury or re-injury inherent in the use of any product of this type.

ITEM NAME: FJ270 MUMFIT BELT BRACE PREGNANCY FJ280 MUMFIT LUMBOSACRAL BRACE PREGNANCY

PURPOSE: Relieves lumbar and pelvic pain during pregnancy. Provides constant support and support to the belly. Elevates the abdomen, improving sacroiliac posture. Strengthens the lower abdominal area. Does not press on the abdominal organs. Does not restrict mobility

INDICATIONS: FJ270: Lumbar, sacroiliac or pelvic pain and discomfort during pregnancy. Pelvic floor weakness. Pelvic organ prolapse (POP). Stress urinary incontinence. FJ280: Low back, sacroiliac or pelvic pain and discomfort during pregnancy.

DESCRIPTION AND CHARACTERISTICS: FJ270: Sacroiliac belt orthosis made of elastic fabric. It incorporates a removable elastic band to increase its support function. Incorporates silicone inserts to prevent displacement. Velcro fastening and adjustment. Length 100 cm. Height 8 cm.

FJ280: Lumbosacral orthosis for pregnancy made of breathable elastic fabric. It incorporates a removable elastic band to increase its support and fastening function. Incorporates flexible lumbar stays. The front design allows an effective adaptation to the changes in size and shape of the belly during the stages of pregnancy. Velcro fastening and adjustment. Silicone lumbar pads. Height 13 cm.

BRACE LENGTH / SIZE SELECTION One size

MATERIALS FJ270: Polyamide 75%, elastane 22.6%, silicone 2.4%. FJ280: Polyamide 44%, elastane 33%, polyester 23%.

INSTRUCTIONS FOR USE AND FITTING: Correct sizing is essential to achieve maximum treatment efficacy as well as prolonging the life of the brace. FJ270: Place the brace over the pelvic area. Surround the abdomen with the

belt by fastening the velcro straps until the desired degree of support is achieved. The front area must be positioned below the abdomen so that it fulfils its function of support.

Subsequently, adjust the degree of compression and support by pulling the side tensioners forward at the same time. FJ280: Place the orthosis over the lumbosacral region. The orthosis is composed of two parts: a lumbosacral back piece and a front piece with Velcro fastening and fixation. Locate the centre of the front part marked "EMO". Attach one end of the front piece to the lumbosacral piece.

Wrap the brace around the abdomen and attach the other end of the back piece to the front piece. Subsequently, adjust the degree of compression and support by pulling the side tensioners forward at the same time and fastening them on the front piece. The front part must be positioned below the belly in order to fulfil its supporting function. The user must regularly inspect the product to ensure that it is in perfect condition and that it is in proper working order. Do not use the product if it is damaged. The intervention of third parties in the installation is not necessary except in cases of limited strength or coordination. In order for the product to maintain its compression or support properties, the orthosis must adapt to the affected area without causing discomfort or discomfort during use. Excessive compression may cause intolerance to the treatment.

IMPORTANT INFORMATION Please read all warnings and instructions before use. Correct application is essential for the product to work properly and to reduce the risk of injury or re-injury inherent in the use of any product.

PRECAUTIONS (A) If any allergic reaction, irritation, skin alteration, swelling or sensitisation occurs, remove the product and consult the prescribing physician or health care professional. Do not use directly on open or reddened scars without proper protection.

Do not expose to a flame or heat source at very high temperatures. The brace should fit snugly but should not compress excessively. Inspect the fit of the product to ensure that it fits properly. Do not over-tighten. Check regularly to ensure that circulation is not compromised. If in doubt as to size, always choose the larger size.

The user should regularly inspect the product to ensure that it is in perfect condition and that it is working properly. Do not use the product if it is damaged.

WARNINGS The use of the orthosis is conditional on the indications. It is recommended that the orthosis be used by one patient only. For correct fitting of the orthosis, follow the manufacturer's instructions. Follow the indications or recommendations of the prescriber or healthcare professional. For a custom fitting,

consult a specialist. The specialist will provide the necessary information to ensure that the end user or the person responsible for fitting the product understands how to use the product safely. Product not suitable for sterilisation. Keep out of reach of children. For disposal of the product use the solid waste container. For disposal of the packaging, use the paper/cardboard container or the plastic container. In all cases, the country's regulations must be strictly complied with.

Any serious incident involving the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or the patient are established.

WASHING, MAINTENANCE AND STORAGE Wash regularly with lukewarm water (maximum 30°), neutral soap and allow to air dry. During washing, if the garment has velcro fasteners, these should be glued together or kept closed on the garment. Do not expose to direct heat sources such as hot surfaces, direct sunlight or other sources of ignition. The orthosis should be checked to ensure that it is thoroughly dry before storage, use and re-use. Contact with ointments, creams, alcohols and solvents may deteriorate the garment and shorten its useful life. Store at room temperature, preferably in the original packaging.

SHelf LIFE, QUALITY CERTIFICATES AND GUARANTEES ESPECIALIDADES MEDICO-ORTOPÉDICAS, S.L. states that should you detect any defect within the guarantee period, please contact your distributor in order to solve the problem as soon as possible. The guarantee will not be valid if the product has been damaged by improper use, or if the instructions and recommendations for use explained in this brochure have not been followed.

IMPORTANT! Please do not forget to keep with date of purchase and stamped by the seller) and present the complete product (case and body of the product), these requirements will be essential for any claim.

With regular maintenance, the product will be reliable and durable and can be reused following the indicated measures for 24 months, after this period of time, it is recommended to change to a new brace.

QUALITY CERTIFICATES: All materials used have been tested, comply with the applicable European standards and offer maximum safety and comfort during use. Oeko-Tex Standard 100 certificate: The Oeko-Tex label guarantees the absence of substances harmful to health and certifies the absence of harmful substances in all materials from which the orthosis is made. Guarantee within the framework of the legal regulations.

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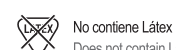
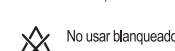
Reciclado Recycled


Mantener seco Keep dry

Mantener alejado de la luz Keep away from light

Reciclable Recyclable

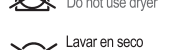Punto verde Green point

No usar blanqueador Do not use whitener

No usar secadora Do not use dryer

Lavar en seco Clean dry