Guante de agarre  
Grip glove  
**GDA55**

00001

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EN GDA55

Dear customer, thank you for placing your trust in an EMO product. By choosing our line of orthoses, you have purchased a range of quality products of the highest sanitary standard. Please read these instructions carefully. If you have any questions, please contact your doctor or specialist retailer.

**IMPORTANT INFORMATION**  
Read all warnings and instructions before using this product. It is essential to apply it correctly for it to function properly and to reduce the risk of injury or recurrence of injuries inherent in the use of any product of this type.

**ITEM NAME**  
Commercial Name: GDA55 GRIP GLOVE

**DESCRIPTION AND CHARACTERISTICS**  
Orthosis made of viscoelastic material and reinforced with non-elastic straps of high resistance. The glove helps to shape the hand with the proper grip and helps to firmly hold a wide variety of objects. Designed for one-handed placement. Indicated for individuals with reduced hand functionality or strength: Helps in recovery after a stroke, cerebral palsy, spinal cord injury, or any disability affecting hand functionality. Suitable for activities of daily living, sports, or work environments. Functional rehabilitation. Helps to grip or hold objects with cylindrical shape or handle such as tools, gym equipment, weights, handlebars, etc.

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**WASHING AND MAINTENANCE**  
Periodically wash with a damp cloth and allow to air dry. Do not expose to direct heat sources such as hot surfaces, direct sunlight, and other sources of ignition. Contact with ointments, creams, alcohols, and solvents may deteriorate the garment and reduce its lifespan. Store at room temperature.

**SIZES**  
Wrist Circumference  
S: 13 - 15 cm; M: 14-18 cm; L: 18 - 22 cm

**MATERIALS**  
Exterior: PA. Internal: Neoprene. Strap: PE

**PLACEMENT**  
Release both Velcro straps. Place the glove over the dorsal area of the hand. Wrist fixation: pass the strap through the plastic loop and firmly fasten around the wrist. Hand strap fixation: pass the strap through the plastic loop and fasten onto itself. Both straps have a terminal to facilitate one-handed placement.

**PRECAUTIONS**  
If any allergic reaction, irritation, skin alteration, swelling, or sensitization occurs, discontinue use of the product. Do not use directly on open or reddened scars without proper protection. Do not expose to flames or high heat sources. The user should regularly inspect the product to ensure it is in perfect condition and guarantee its proper functioning. Do not use the product if it is damaged.

**WARNINGS**  
The use of this device is subject to its indicated use. Keep out of reach of children. For product disposal, use the solid waste container. For container disposal, depending on the material, use the paper/cardboard container or the plastic container. In any case, strictly comply with the country's regulations.

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